

**SAMPLE
FILLING CARD**

PLEASE PAY ATTENTION TO THE CORRESPONDING UNYVERO APPLICATION MANUAL
AND RESPECTIVE SAMPLE HANDLING CARD

BLOOD CULTURE

BCU, IAI



Positive blood culture

50 µl



LIQUID SAMPLES

HPN, ITI, IAI



max.
volume

BAL, synovial fluid, ascites fluid, ...

180 µl



SOLID ORGANIC SAMPLES

ITI, IAI



max.
volume

Tissue, bone fragments, ...

3 x 3 x 3 mm (27 mm³)

Fill up with up to 150 µl buffer from
the Unyvero Pre-Treatment Tool.

